

# CompreBoot Sizing Chart

## MEASURING INSTRUCTIONS:

- 1: Record **length** measurement **K** (length from heel to 1st metatarsal head, or desired length) on the corresponding line. Select the boot size that matches according to **row K** on the sizing chart.
- 1b: (**MedaBoot only**) Record **length** measurements **M** (top of foot/3rd Matatarsal Head to Ankle Bend) on the corresponding lines.
- 2: Record **circumference** measurements **I**, **J**, and **L** on the corresponding lines.
- 3: Check to make certain that foot measurements **I**, **J**, and **L** do **not exceed** the adjustment range of the boot size selected in **Step 1** (measurement **K** - sizing chart **row K**).
- 4: If any of the above foot measurements (**I**, **J**, and **L**) exceed the adjustment range of the boot size selected in **Step 1** (**K**) above, select the **next larger size** where all three measurements do not exceed the adjustment range of the boot.

